





### Step One

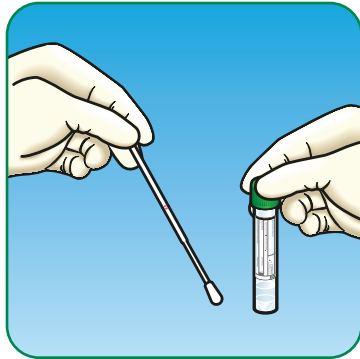



- **PEEL** open pouch and remove swab. Avoid touching the sterile area between the swab tip and breakpoint.
- Collect a small amount of stool by rotating and covering the swab tip with feces. **Do not use the swab as a tool to scoop sample into the tube.**

### Step Two

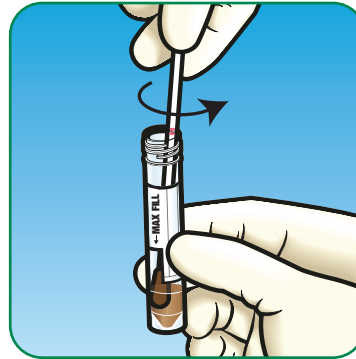

- **OPEN TUBE** and Insert swab in tube. **DO NOT POUR OUT LIQUID. DO NOT EXCEED FILL LINE INDICATED ON THE TUBE LABEL.**
- **MASH AND MIX** stool specimen against the side of the tube to disperse specimen into the medium.

### Step Three

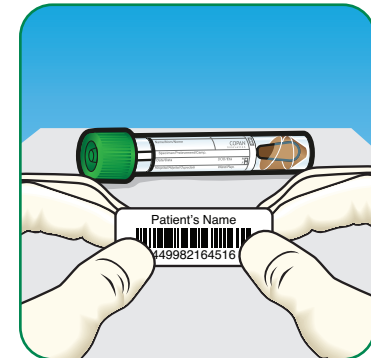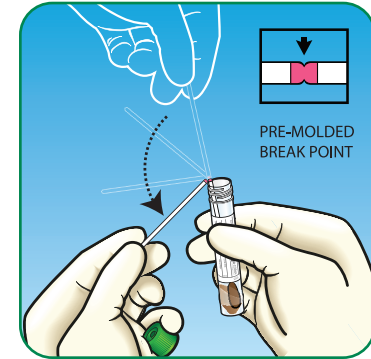

- Holding the tube away from your face **BEND AND TWIST** the swab applicator at visible molded breakpoint and discard the upper shaft.
- **TWIST** cap tube to tighten and shake until mixed.
- **LABEL** container with barcode. Avoid covering the **MAX FILL** line.

**After collection, please store at room temperature (20 – 25°C) and deliver to the laboratory within 48 hours.**